


FlexCell Bend Radius *for the prosthetist*

www.i-biomed.com

info@i-biomed.com

443-451-7175


FlexCell Battery Dimensional Axes

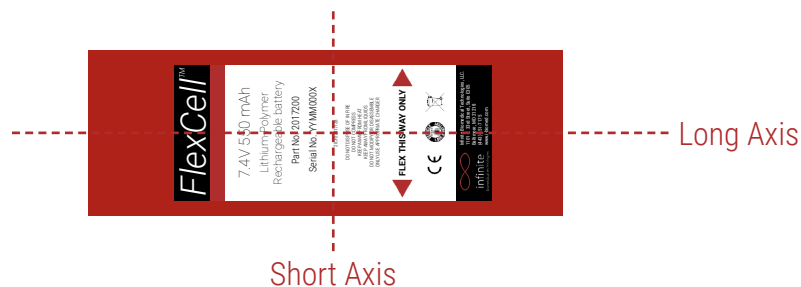

Pictured below is the maximum bend radius for both the long and short axis of a FlexCell battery. The maximum bend curvature of the FlexCell battery cannot exceed the curvature depicted below for each axis.




The FlexCell 550 mAh battery is not designed to bend around the short axis; doing so can damage the battery and void the warranty. Bend 550 mAh batteries around the long axis only.

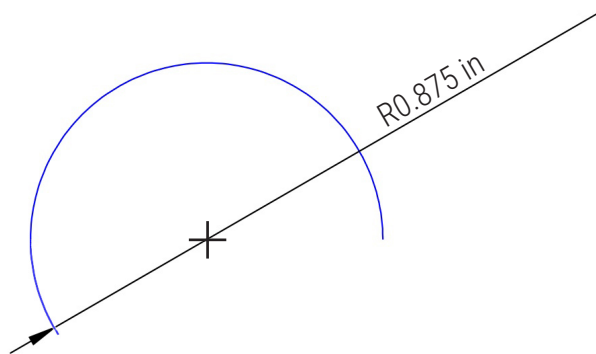

Long Axis

**FOR FLEXCELL
AND FLEXCELL MINI**

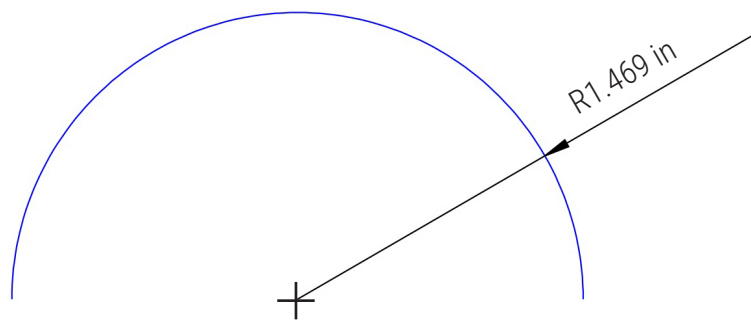


Short Axis

FOR FLEXCELL MINI ONLY



Example: FlexCell Long Axis

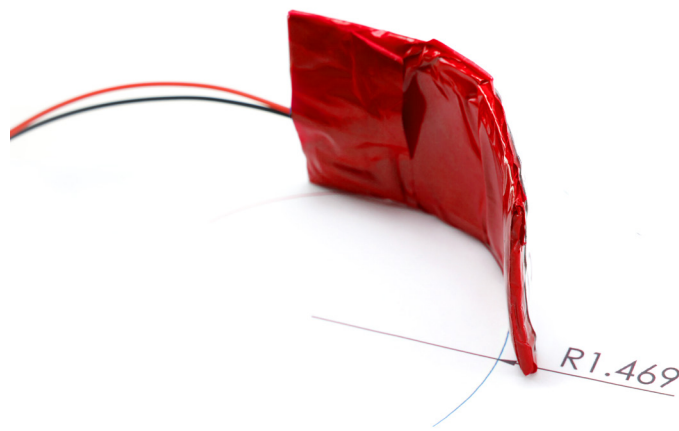

Example: FlexCell Mini Short Axis